

ODYSSEY
Infrared Listening System
User Instructions

This headset will assist your hearing aids. You do not have to remove them.

The headset will work as a regular headset, or as a magnetic loop system.

- Place the headset over your ears and your hearing aids (if wearing) with the headband over the top of your head. Adjust the ear-pieces up and down as needed for a good fit.
- Turn on the headset with the on the right ear. With this same knob, adjust the volume to a comfortable level. The two lights below the volume control should be on.
- If the sound is not loud enough, you may correct by doing one or more of the following:
 - Adjust the volume on the headset with the same knob used to turn on the headset.
 - Adjust the volume level of your hearing aids if possible.
 - Move the switch on your hearing aids to the “Telecoil” position if available.
 - On completion of the program please return the headset to the same check-out station.

Thank you for using the Odyssey System